

A Denture Reline Material Has to Do Two Things Well: Bond to the Denture and Stay Soft.

StaySoft® S.R.M

Soft Reline Material

StaySoft S.R.M. Soft Reline Material is the chairside reline material that does both. Its unique adhesive system tenaciously bonds to denture acrylic (you can't peel it off) and its scientifically formulated reline material contains no plasticizers that might leach out and cause it to harden. In other words, it stays on and stays soft.

And *StaySoft S.R.M.* Soft Reline Material is extremely easy to use. Just prepare the denture as usual and apply the adhesive, then apply the soft reline material from its automix cartridge. *StaySoft S.R.M.* Soft Reline Material is a silicone, not an acrylic, so there is no mixing, no odors and no awful chemical taste.

For denture relines, choose *StaySoft S.R.M.* Soft Reline Material. It performs reliably, bonds tenaciously and stays soft.



- > Convenient, one-visit chairside system
- > Contains no plasticizers, so it stays soft longer
- > Convenient cartridge dispensing eliminates messy mixing
- > Silicone-based liner produces no heat during setting for improved patient comfort
- > Hydrophilic liner lowers water sorption - less staining, less odors
- > Two shades of liner available - Light and Dark Gingiva
- > Ideal system for relief of pressure areas, for cushioning implants and for overdentures
- > Sets in the mouth in under 3 minutes with no unpleasant taste



StaySoft® S.R.M

Soft Reline Material

Physical Properties

Stay Soft S.R.M. Liner

End Hardness:	= 30 Shore A
Tensile Strength:	> 3.0 N/mm ² (squared)
Strain in Compression:	6.5 - 7.5%
Working Time:	1 - 1:20 min.
Setting Time:	3:30 min.

Stay Soft S.R.M. Varnish

End Hardness:	= 25 Shore A
Tensile Strength:	> 0.5 N/mm ² (squared)
Strain in Compression:	10 - 13%
Working Time:	1 - 1:20 min.
Setting Time:	3:30 min.

Ordering Information:

Stay Soft S.R.M. Intro Kit Gingival Light

REF N37

Stay Soft S.R.M. Intro Kit Gingival Dark

REF N37D

- 1 - 50ml Cartridge Liner (either Light or Dark)
- 1 - 4ml bottle Primer
- 1 - 5ml bottle Varnish Base
- 1 - 5ml bottle Varnish Catalyst
- 1 - Mixing Well
- 6 - Mixing tips
- 20 - Disposable Applicators
- Instructions

Refill Kits:

Refill Kits includes 1 - 50ml Cartridge Liner (either Light or Dark)

1 - 4ml bottle Primer and 6 - Mixing tips

1 -50ml Cartridge Liner Gingival Light N37A

1 - 50ml Cartridge Liner Gingival Dark N37B

1 - 5ml bottle each of Base & Catalyst- Varnish N37C

24 - Mixing tips N37E

1 - 4ml bottle of Primer N37AA

1 - Dispenser Gun Q19

For more information on our entire line of innovative dental products visit us at **www.pentron.com** or call us at **1.800.551.0283**

800.551.0283 | www.pentron.com | 203.265.7397

FibreKor® Post | Build-It® FR™ | Stay-Soft SRM® | Crown & Bridge | Impressions | Adhesives | Restoratives

Subject to change or cancellation without notice. Copyright © 2010 Pentron Clinical. All rights reserved.

VLT_Rev02

 **Pentron
Clinical**
Sybron Dental Specialties