FibreKleer® 4X™

Fiber Post

Instructions for use

English

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Srpski

Hrvatski

Norsk

Español

Français

Português

Türkçe

800.551.0283 | 203-265-7397 | www.pentron.com

FibreKleer® 4X™ posts feature longitudinal glass fibers bundled in a strong resin matrix to deliver the ultimate in strength, flexural modulus, and esthetics that are not attainable with traditional metal posts. Each *FibreKleer 4X* posts is equivalent to ~370% of aluminum to facilitate identification on radiographs. The drills and posts are color coded to streamline the size selection process. Available in three body designs, parallel with a retentive head, tapered, and original, the *FibreKleer 4X* Post allows clinicians to choose the type of post that best suits each individual case.

Indications:

- Retaining core build-ups in endodontically treated teeth that have a minimum of 2mm of coronal tooth structure above the crestal bone

Contraindications:

- *FibreKleer 4X* posts are contraindicated in teeth with less than 2mm of coronal tooth structure above the crestal bone
- It is contraindicated to lute a *FibreKleer 4X* post with glass ionomer, zinc phosphate or polycarboxylate cement

Please observe the following important guidelines when placing *FibreKleer 4X* Posts:

- *FibreKleer 4X* posts are pre-silanated for increased adhesion to resin cements and core

materials.

- If 2mm of coronal tooth structure above the crestal bone is unavailable, crown lengthening or orthodontic forced eruption should be considered to achieve adequate ferrule.
- To minimize stress on the core-tooth interface and the post contained within, the finish line or margin of the preparation should remain on solid tooth structure, not on the core buildup.
- Resin cements and core materials provide the best result when used with the *FibreKleer 4X* Post System. Pentron Clinical recommends Cement-It® Universal C&B™ Resin Cement, Breeze® Self-Adhesive Resin Cement, Build-It® FR™ Fiber Reinforced Core Build-up Material, and Build-It® Light Cure Core Material.
- *FibreKleer 4X* Posts must be disinfected. Cold sterilants such as glutaraldehyde or phenols may be used. Following disinfection thoroughly rinse and dry the post before cementing.
- *FibreKleer* and *FibreKor*® Drills must be sterilized via steam autoclaving.
- *FibreKleer 4X* Posts must be cut with a diamond bur or a diamond separating disk and it is recommended to avoid cutting fiber posts with a carbide bur as this may splinter the post.
- Cement may be carried to the canal with a paste filler to ensure complete coverage of cement in the canal.
- Use of the intra-oral tip on the auto-mix syringe help avoid air entrapment in the post space.

- If endodontic retreatment of the apex is required, the *FibreKleer 4X* Post can be removed by drilling into the center of the post with a needle diamond to a depth of 2-3mm to start then continue with the corresponding size of *FibreKleer* or *FibreKor* Drill. It is recommended that during post removal frequent radiographs are taken to ensure that the instruments are following the path of the post to help prevent perforation of the root.

Instructions for Placing the *FibreKleer 4X* Post

Canal Preparation

1. Select a post with a diameter that is slightly wider than the diameter of the obturated canal at the tooth's cervical to ensure preservation of as much cervical tooth structure as possible.
2. Using the drill, determine the depth of the post space preparation, leaving at least 4mm of Resilon**obturation material or gutta percha at the apical end. Place rubber endodontic stops on the drills corresponding to the desired depth to avoid overpreparing the post space. The post must extend 3-5mm apical to the crestal bone maintaining 4mm (minimum) of apical obturation material.
3. Use a heated instrument to remove the obturation material followed by a narrower post drill to the desired depth.
4. Using the *FibreKleer* or *FibreKor* Drill that matches the post to be inserted complete the canal preparation.

5. Flush out the canal to remove the debris and dry it with an air syringe and paper points.
6. Trial seat the *FibreKleer 4X* Post in the canal that corresponds to the size of the last drill used. A radiograph may be taken at this point to verify that the post is fully seated.
7. *FibreKleer 4X* Post must be trimmed with a watercooled, high-speed diamond separating disk. Rotate the post while cutting to ensure a smooth cut. Trimming of the *FibreKleer 4X* Parallel Posts should be done from the apical end to preserve the retentive head feature. *FibreKleer 4X* Tapered Posts should be trimmed from the occlusal end to preserve the apical taper.
8. Clean the post with an alcohol wipe and dry it with a blast of oil-free air.

Post Cementation

9. — **If a total-etch technique is preferred:**
 - a. Acid-etch the canal with 37% phosphoric acid for 20 seconds.
 - b. Rinse the canal thoroughly and dry using paper points. Leave the dentin slightly moist.
 - c. Introduce Bond-1® Primer/Adhesive into the canal using a syringe or a Microbrush. If a Microbrush is used, two or more applications may be required. Gently dry the canal with a stream of oil-free air and use a paper point to remove excess resin from the canal. Do not light cure at this point. Please see the instructions for use for *Bond-1* Primer/Adhesive.

- d. Introduce *Cement-It Universal C&B Cement* or *Build-It FR Core Build-up Material* into the canal with a paste filler or by injecting it directly into the canal with an intra-oral auto-mix tip.
- e. Coat the Post *Cement-It Universal C&B Cement* or *Build-It FR Core Build-up Material* and insert the post into the canal with gentle finger pressure, allowing excess cement to vent. Remove the excess cement with a plastic instrument. Please see the instructions for use for the cement or core material work/set times.

— **If a self-adhesive technique is preferred:**

- a. Introduce *Breeze Self-Adhesive Resin Cement* into the post space using the intra-oral auto-mix tip.
 - b. Coat the Post with *Breeze Self-Adhesive Resin Cement* and insert the post into the canal with gentle finger pressure, allowing excess cement to vent. Remove the excess cement with a plastic instrument. Please see the instructions for use for *Breeze Self-Adhesive Resin Cement* work/set times.
10. Initiate the set of the dual cure cement or core material by light curing the top of the canal for 10 seconds. Consult the instructions for use for the *Breeze Self-Adhesive Resin Cement*, *Cement-It Universal C&B Cement* or *Build-It FR Core Build-up Material* work/set times.
 11. Using a matrix band or a *Build-It® Core Form*, fabricate a build-up by bonding *Build-It*

FR Core Build-up Material or *Build-It® Light Cure Core Material* to the post and remaining dentin.

12. Shape the cured core material to receive the final restoration using diamond and carbide burs.

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Limited Warranty

Express Limited Warranty:

Pentron Clinical warrants that this product is not defective for a period of one year.

Buyer's Exclusive Remedy:

In the event that this product proves to be defective within a period of one (1) year from the date of shipment to Buyer, Buyer's exclusive remedy shall be at Pentron Clinical's option of either replacement of the product or defective part or component, free of charge, or refund of the purchase price. Buyer is only entitled to this remedy upon showing that the product has been stored and used in accordance with Pentron Clinical's instructions with the exception that this warranty nevertheless extends to products whose expiration dates fall within the one-year limited warranty period.

Disclaimer of Consequential Damages:

Pentron Clinical hereby disclaims any liability for loss or damage, direct or consequential, that result from the use of this product.

Disclaimer of All Other Warranties:

THERE ARE NO OTHER WARRANTIES OF ANY KIND, EXPRESS OR IMPLIED, CONNECTED WITH THE SALE OF THIS PRODUCT. Pentron Clinical SPECIFICALLY DISCLAIMS ANY IMPLIED WARRANTIES OF MERCHANTABILITY AND FITNESS FOR A PARTICULAR PURPOSE.