|   |            |
|---|------------|
| <b><u>Hrvatski/Croatian</u></b> . . . . .     | <b>p2</b>  |
| <b><u>Česky/Czech</u></b> . . . . .           | <b>p3</b>  |
| <b><u>Dansk/Danish</u></b> . . . . .          | <b>p4</b>  |
| <b><u>Nederlands/Dutch</u></b> . . . . .      | <b>p5</b>  |
| <b><u>English</u></b> . . . . .               | <b>p6</b>  |
| <b><u>Eesti/Estonian</u></b> . . . . .        | <b>p7</b>  |
| <b><u>Suomi/Finnish</u></b> . . . . .         | <b>p8</b>  |
| <b><u>Français/French</u></b> . . . . .       | <b>p9</b>  |
| <b><u>Deutsch/German</u></b> . . . . .        | <b>p10</b> |
| <b><u>Elinika/Greek</u></b> . . . . .         | <b>p11</b> |
| <b><u>Magyar/Hungarian</u></b> . . . . .      | <b>p12</b> |
| <b><u>Íslenska/Icelandic</u></b> . . . . .    | <b>p13</b> |
| <b><u>Italiano/Italian</u></b> . . . . .      | <b>p14</b> |
| <b><u>Lietuviška/Lithuanian</u></b> . . . . . | <b>p15</b> |
| <b><u>Norsk/Norwegian</u></b> . . . . .       | <b>p16</b> |
| <b><u>Polski/Polish</u></b> . . . . .         | <b>p17</b> |
| <b><u>Português/Portuguese</u></b> . . . . .  | <b>p18</b> |
| <b><u>Română/Romanian</u></b> . . . . .       | <b>p19</b> |
| <b><u>Srpski/Serbian</u></b> . . . . .        | <b>p20</b> |
| <b><u>Slovenčina/Slovakian</u></b> . . . . .  | <b>p21</b> |
| <b><u>Slovenščina/Slovenian</u></b> . . . . . | <b>p22</b> |
| <b><u>Español/Spanish</u></b> . . . . .       | <b>p23</b> |
| <b><u>Swenska/Swedish</u></b> . . . . .       | <b>p24</b> |
| <b><u>Türkçe/Turkish</u></b> . . . . .        | <b>p25</b> |

***Finí*<sup>TM</sup> sustav diskova za završne radove i poliranje**

**Preporuke za uporabu *Finí* diskova**

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**Finí™ Finishing and Polishing Disk System**

**Guidelines for Using *Finí* Disks**

1. Do not use the *Finí* System on handpieces capable of speeds greater than 25,000 rpm.
2. The polishing motion should be light, intermittent, unidirectional strokes.
3. Keep both the disk and restoration surface dry when polishing as this will produce a more uniform, smoother appearance.
4. Sterilize the mandrels in an autoclave between each use.

**Finishing and Polishing Composite and Resin Ionomer Restorations**

1. Attach the selected *Finí* Disk to the mandrel by inserting the mandrel completely into the rubber collar until the mandrel reaches the bottom. To remove the disk, lightly pinch the rubber collar with your index finger and thumb and remove the disk from the mandrel.
2. Remove the excess material from the restoration and contour the desired shape using your instruments of choice.
3. Begin with a *Finí* Coarse Disk (black) at 15,000 rpm for gross reduction. Lubrication is not recommended during the finishing and polishing process. Rinse and dry the restoration after gross reduction has been completed.
4. Follow with a *Finí* Medium Disk (blue) at 10,000 - 15,000 rpm for final contouring. This should take approximately 15 to 20 seconds.
5. Final polishing is accomplished by first using the *Finí* Fine Disks (red) for 10 seconds at 15,000 rpm. Follow this with the *Finí* Extra-Fine Disks (white) at 25,000 rpm with feather-like strokes for 10 to 15 seconds. This sequence will achieve a brilliant shine that will rival natural esthetics.

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