<u>Hrvatski/Croatian</u> p2
<u>Česky/Czech</u> p3
<u>Dansk/Danish</u> p4
<u>Nederlands/Dutch</u> p5
<u>English</u> p6
<u>Eesti/Estonian</u> p7
<u>Suomi/Finnish</u> p8
<u>Français/French</u> p9
<u>Deutsch/German</u> p10
<u>Elinika/Greek</u> p11
<u>Magyar/Hungarian</u> p12
<u>Íslenska/Icelandic</u> p13
<u>Italiano/Italian</u> p14
<u>Lietuviška/Lithuanian</u> p15
<u>Norsk/Norwegian</u> p16
<u>Polski/Polish</u> p17
<u>Português/Portuguese</u> p18
<u>Română/Romanian</u> p19
<u>Srpski/Serbian</u> p20
<u>Slovenčina/Slovakian</u> p21
<u>Slovenščina/Slovenian</u> p22
<u>Español/Spanish</u> p23
<u>Swenska/Swedish</u> p24
<u>Türkçe/Turkish</u> p25

UPUTE ZA UPORABU

First-Fill je materijal za privremene restorativne ispune koji se stvrdnjava svjetlom i koji je pogodan za privremene restoracije, za onleje i inleje kao i podlogu za montažne privremene krune i mostove napravljene od metakrilatnih i polikarbonatnih materijala.

First-Fill materijal za privremene ispune ostaje elastičan nakon stvrdnjavanja svjetlom. Odstranjivanje je brzo i jednostavno čak ako postoje i manji zastoji pri pripravljanju. *First-Fill* materijal za privremenu ispunu eliminira oštećenja na marginama pripravka.

Upute za uporabu *First-Fill* materijala za privremene ispune:

1. Izolirajte zub pamučnim tamponima kako biste osigurali da je zub suh (klinovi i matrica se mogu primijeniti ako je to potrebno).
2. Ako je potrebno očistite priprevak zračnom/vodenom štrcaljkom i osušite.
3. Stavite traženu količinu *First-Fill* materijala za privremene ispune na pločicu za miješanje i pokrijte dok nije spremna za upotrebu.
4. Postavite materijal u kavitet pripravka i oblikujte ga spatulom ili drugim tupim instrumentom. Ako se materijal lijepi za instrument, malo ovlažite instrument sa sredstvom za vezivanje (npr. Bond-1™).
5. Uz pomoć svjetlosnog pištolja stvrdnjavajte *First-Fill* u trajanju od 40 sekundi. Veše restoracije mogu tražiti dodatno stvrdnjavanje svjetlom 10 sekundi po svakoj sekciji. Materijal ima dubinu stvrdnjavanja od 5 mm.
6. Odstranite matricu i provjerite okluziju.
7. *First-Fill* materijal za privremenu ispunu se može podesiti silikonskim gumenim vršcima ili okruglim svrdlima. Višak se može odstraniti oštricom skalpela.
8. *First-Fill* materijal za privremenu ispunu obično ostaje na svom mjestu kod standardnih inlej pripravaka. Ako je potrebno dodatno pridržavanje, *First-Fill* materijal za privremenu ispunu se može cementirati uz pomoć cementa bez eugenola.
9. Ako je potrebno, kompozit koji se stvrdnjava svjetlom može se koristiti preko *First-Fill* materijala za privremenu ispunu za okluzalni dio. Svjetlom stvrdnjavajte kompozit zajedno ili odvojeno od *First-Fill* sloja materijala za privremenu ispunu.

MJERE OPREZA:

First-Fill materijal za privremenu ispunu se treba čuvati na sobnoj temperaturi (20°C). Čuvajte na hladnom mjestu veše količine dok ih ne budete trebali. Ne izlažite svjetlu. - Materijal je osjetljiv na svjetlost. *First-Fill* materijal za privremenu ispunu ima rok trajanja od dvije godine. Ne koristite nakon isteka roka trajanja.

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INSTRUCTIONS FOR USE:

First-Fill® Material is a light cure temporary restorative filling material suited for temporary restorations; for onlays and inlays and relines material for prefabricated temporary crowns and bridges made of methacrylate and polycarbonate materials.

First-Fill Temporary Filling Material remains elastic after curing. Removal is quick and simple-even if there are minor undercuts to the preparation. *First-Fill* temporary filling material eliminates damage to the preparation's margins.

Instructions for using *First-Fill* temporary filling material:

1. Isolate tooth with cotton rolls to ensure tooth is dry (wedges and a matrix may be applied if necessary).
2. Clean preparation with air/water syringe if necessary and dry.
3. Place desired amount amount of *First-Fill* temporary filling material onto mixing pad and cover until ready to use.
4. Manipulate material into cavity preparation and contour with a spatula or any blunted instrument. If material sticks to instrument, slightly wet instrument with light cure bonding material (e.g. Bond-1® bonding agent).
5. Light cure *First-Fill* for 40 seconds with light gun. Larger restorations may require additional light curing, 40 seconds per section. Material has a 5mm depth of cure.
6. Remove matrix and check occlusion.
7. *First-Fill* temporary filling material may be adjusted with silicone rubber points or round burs. Excess may also be removed with a scalpel blade.
8. *First-Fill* temporary filling material usually stays in place with standard inlay preparations. If additional retention is necessary, *First-Fill* temporary filling material may be cemented with a eugenol-free cement.
9. If desired, a light cured composite may be used over *First-Fill* temporary filling material for the occlusal portion of the temporary. Light cure the composite together or separate from the *First-Fill* temporary filling material layer.

PRECAUTIONS:

First-Fill temporary filling material should be stored at room temperature (74°F or 22°C). Refrigerate larger quantities until needed. Do not expose to light. Material is light sensitive. *First-Fill* temporary filling material has a 2 yr. shelf life. Do not use after expiration date.

If using a light cured base or lining materials in the cavity preparation, isolate them with Vaseline or other separating medium to prevent *First-Fill* temporary filling material from bonding to them.

First-Fill temporary filling material is not suitable for full crowns and bridges. Do not use zinc oxide eugenol cements or materials in conjunction with *First-Fill* temporary filling material as they may inhibit the polymerization of *First-Fill* temporary filling material.

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