**Hrvatski/Croatian . . . . . p2**  
**Česky/Czech . . . . . p4**  
**Dansk/Danish . . . . . p6**  
**Nederlands/Dutch . . . . . p8**  
**English . . . . . p10**  
**Eesti/Estonian . . . . . p12**  
**Suomi/Finnish . . . . . p14**  
**Français/French . . . . . p16**  
**Deutsch/German . . . . . p18**  
**Elinika/Greek . . . . . p20**  
**Magyar/Hungarian . . . . . p22**  
**Íslenska/Icelandic . . . . . p24**  
**Italiano/Italian . . . . . p26**  
**Lietuviška/Lithuanian . . . . . p28**  
**Norsk/Norwegian . . . . . p30**  
**Polski/Polish . . . . . p32**  
**Português/Portuguese . . . . . p34**  
**Română/Romanian . . . . . p36**  
**Srpski/Serbian . . . . . p38**  
**Slovenčina/Slovakian . . . . . p40**  
**Slovenščina/Slovenian . . . . . p42**  
**Español/Spanish . . . . . p44**  
**Swenska/Swedish . . . . . p46**  
**Türkçe/Turkish . . . . . p48**

**UPUTE ZA UPORABU:**
**Upute za uporabu tekućeg dentina Fusio™**
**Indikacije:**

- samoljepiva osnovna podloga ispod restauracijskog materijala visoke viskoznosti
- restoracije klase III
- restoracije klase V
- restoracija malih šupljina klase I
- blokiranje podrezivanja
- pečačenje jamica i fisura

**Kontraindikacije:**

- samoljepivi tekući kompozit Fusio™ ne stavljajte na područja izložena naprezanjima. Na tim područjima primijenite nano kompozit Artiste®.
- Ne stavljajte gravuru sa fosforom kiselinom na dentin prije nego ste stavili Fusio™ tekući kompozit.
- Ne preporučuje se za direktno prekrivanje.
- **Upozorenje:** Uporabu ovog proizvoda treba izbjegavati kod pacijenata koji su preosjetljivi na bilo koji od njegovih sastojaka.

**Nekompatibilnost**

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### Fusio™ Liquid Dentin Instructions for Use

#### Indications:

- Self-Adhesive base liner under higher viscosity restorative materials
- Class III restorations
- Class V restorations
- Restoration of small Class I cavities
- Blocking out undercuts
- Pit and fissure sealant

#### Contraindications:

- Avoid placing *Fusio* Liquid Dentin in stress bearing areas. Use *Artiste® Nano Composite* for these applications.
- Do not apply phosphoric acid etch to dentin prior to placing *Fusio* Liquid Dentin.
- Not recommended for direct pulp capping.

**Caution:** The use of this product should be avoided on patients with a known sensitivity to any of its ingredients.

#### Incompatibility:

- Avoid using materials that contain eugenol or clove oil as they may inhibit the polymerization of this composite and interfere with bonding to tooth structure. Contact with cationic mouth rinses, plaque disclosing agents and chlorhexidine may result in discoloration.

### GENERAL INSTRUCTIONS FOR USE

#### (Class I, III, V, blocking out undercuts):

1. Clean the teeth before selecting the shade.
2. Isolate the treatment area with a rubber dam.

**Important Note for Class V Preparation with Caries:** Ensure there is a physically present and well defined cavity preparation. Create an outline form removing all caries at DEJ and all decalcified enamel. Place a bevel on all enamel margins.

3. Prepare the cavity according to the principles of adhesive dentistry. Avoid any sharp internal edges. In anterior regions, bevel the enamel edges for better retention. Thoroughly clean the cavity and remove any potential debris with water. Blot or air-dry briefly (2–3 seconds) to remove excess water for a moist surface with no pooling water. The surface should glisten and appear slightly shiny. Do not desiccate the tooth.

**Important Note:** Ensure that surface debris is removed and the preparation is clean. Intimate contact of *Fusio* Liquid Dentin to the tooth surface is essential for achieving a good and reliable bond.

4. For deep cavities cover the floor adjacent to the pulp with calcium hydroxide material first and subsequently use *Fusio* Liquid Dentin.
5. Use a transparent matrix for cavities affecting the proximal areas. In the posterior region, a stainless steel matrix may be used with caution as the flowable may adhere to the matrix band.
6. Using the bent needle tip supplied, dispense a 1mm increment of *Fusio* Liquid Dentin directly onto the tooth surface. Gently rub the material into the preparation using the needle tip for 20 seconds. A *Microbrush®* micro applicator may also be used to agitate the initial layer of *Fusio* Liquid Dentin for 20 seconds. Pull the syringe plunger back after use to prevent overflow of the *Fusio* Liquid Dentin from the syringe tip.
7. Light cure the initial layer for 10 seconds with a curing light with output in excess of 600 mW/cm<sup>2</sup>.
8. Syringe additional *Fusio* Liquid Dentin as necessary. Do not apply in increments of more than 2mm thickness for proper light curing for the subsequent layer(s). Light cure each additional 2mm increment for 10 seconds. The final layer should be cured for an additional 10 seconds.
9. Remove excess material with a *Fini™* Polishing Disk or a fine grain diamond. Remove the proximal excess with diamond or tungsten carbide finishers or finishing strips. Check the occlusion and adjust it as necessary.
10. Polish the restoration to a high gloss using *Fini* Polishing disks.
11. For maximum surface sealing, to reduce surface wear, and to minimize microleakage at the bonding interfaces, following trimming and polishing apply *Protect-It®* Composite Surface Sealant to seal the restoration surface and surrounding tooth surface margins.

(Please see the *Protect-It Composite Surface Sealant* instructions for step by step details.)

(Continued on the next page.)

1717 West Collins Ave.,  
Orange, CA 92867 tel: 203.265.7397

CEpartner4U, 3951DB; 13. NL.  
tel: +31 (0)6.516.536.26

Fusio Web-TIFU\_Rev02

## INSTRUCTIONS FOR USE

### PIT AND FISSURE INSTRUCTIONS FOR USE:

1. Isolate the treatment area with a rubber dam.
2. Prepare the enamel surface with an air abrasion unit following the manufacturer's instructions. If an air abrasion unit is not available use 37% phosphoric acid. Apply the acid etch to the tooth for 20 seconds.
3. Rinse the tooth thoroughly with water. Blot or air-dry briefly (2–3 seconds) to remove excess water for a moist surface with no pooling water. The surface should glisten and appear slightly shiny. Do not desiccate the tooth.
4. Using the bent needle tip supplied, gently rub the material into the preparation using the needle tip for 20 seconds while dispensing *Fusio* Liquid Dentin directly onto the tooth surface. A *Microbrush* micro applicator may also be used to agitate the *Fusio* Liquid Dentin for 20 seconds. Pull the syringe plunger back after use to prevent overflow of the *Fusio* Liquid Dentin from the syringe tip.
5. Light cure for 20 seconds with a curing light with output in excess of 600 mW/cm<sup>2</sup>.
6. Check the occlusion and adjust it as necessary.

### DENTIN REPLACEMENT MATERIAL OR SELF-ADHESIVE BASE LINER INSTRUCTIONS FOR USE

1. Clean the teeth before selecting the shade.
2. Isolate the treatment area with a rubber dam.
3. Prepare the cavity according to the principles of adhesive dentistry. Avoid any sharp internal edges. Remove any potential debris with water and thoroughly clean the cavity. Blot or air-dry briefly (2–3 seconds) to remove excess water and form a moist surface with no water pooling. The surface should glisten and appear slightly shiny, not dull. Do not desiccate the tooth.

**Important Note:** Ensure that surface debris is removed and the preparation is clean. Intimate contact between *Fusio* Liquid Dentin and the tooth surface is essential for achieving a good and reliable bond.

4. For deep cavities cover the floor adjacent to the pulp with calcium hydroxide material and subsequently use *Fusio* Liquid Dentin.
5. Use a transparent matrix for cavities affecting the proximal areas. In the posterior region, a stainless steel matrix may be used with caution as the flowable may adhere to the matrix band.
6. Using the bent needle tip supplied, dispense a 1mm increment of *Fusio* Liquid Dentin directly onto the tooth surface. Gently rub the material into the preparation using the needle tip for 20 seconds. A *Microbrush* micro applicator may also be used to agitate the initial layer of *Fusio* Liquid Dentin for 20 seconds. Pull the syringe plunger back after use to prevent overflow of the *Fusio* Liquid Dentin from the syringe tip.
7. Light cure the initial layer for 10 seconds with a curing light with output in excess of 600 mW/cm<sup>2</sup>.
8. Continue to add subsequent layers of *Fusio* Liquid Dentin as desired. Do not apply in increments of more than 2mm thickness for proper light curing for the subsequent layer(s). Light cure each additional 2mm increment for 10 seconds. The final layer should be cured for an additional 10 seconds.
9. Total etch the enamel, any dentin that may have not been covered by the Liquid Dentin, and the *Fusio* Liquid Dentin filled prep with 37% phosphoric acid etchant for 20 seconds. Rinse thoroughly. Remove excess moisture using suction. Leave surface moist.
10. Apply 2 coats of Bond-1® Primer/Adhesive to the enamel and flowable composite. Lightly air thin for 10 seconds to remove the solvent. Light cure for 10 seconds.
11. Apply an *Artiste* Nano Composite enamel shade in 0.5mm to 2mm layers maximum, cure, and polish.

*Microbrush* is a registered trademark of *Microbrush* International.

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